


Refer to the appropriate ARIES® system operation manual and applicable assay package inserts for detailed instructions, precautions, and warnings.

**IVD** For In Vitro Diagnostic Use.



**Luminex Corporation**  
12212 Technology Boulevard  
Austin, Texas 78727-6115  
U.S.A.  
Phone: 512-219-8020  
Fax: 512-219-5195

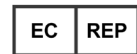

**WMDE**  
Bergerweg 18  
6085 AT Horn  
The Netherlands

**Technical Support**  
Email: [support@luminexcorp.com](mailto:support@luminexcorp.com)  
Phone: 512-381-4397  
Toll-free: 1-877-785-2323  
International: + 800-2939-4959

**ARIES®** Systems Quick Guide  
PN 89-00002-00-508 Rev. B  
May 2016

Luminex Corporation (Luminex) reserves the right to modify its products and services at any time. Notifications will be sent to end users regarding changes that impact the use, performance and /or safety and effectiveness of the device. Any modifications to the device will be made in accordance with applicable regulatory requirements. Luminex assumes no liability for any damages resulting from the off-label application or misuse of this information.

Luminex and ARIES are trademarks of Luminex Corporation, registered in the U.S. and other countries.

All other trademarks, are trademarks of their respective companies.

This product, or use thereof, is covered, in whole or in part, or made by processes covered by one or more patents: [www.luminexcorp.com/patents](http://www.luminexcorp.com/patents).

# ARIES®

## Systems Quick Guide

**Luminex** For a copy of the SDS, visit: [www.luminexcorp.com](http://www.luminexcorp.com)


© 2015 - 2016 Luminex Corporation. All rights reserved. No part of this publication may be reproduced, transmitted, transcribed, or translated into any language or computer language, in any form or by any means without prior express, written consent of Luminex Corporation.

## Creating a New Order

1. Navigate to **Order Management** > **Sample Orders**.



2. Select **New Order** from the Page Action bar.

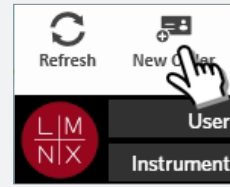

3. Scan or enter the cassette information.



4. Scan or enter the **Sample ID**. If required, enter the **Accession ID** and **Requisition Number**.



5. Select or scan **Save**.




6. You should now see the order listed on the **Sample Orders** page.


Sample ID	Accession ID	Requisition Number	Sample Name	Substrate Type / Assay Name	User	Order Control	Lot Number
Sample 7			Lumina_HIV_AJ0 IA v2 Sample	D- (D) HIV	admin	05/16/2015	
Control 1			Lumina_HIV_AJ0 IA v2 Control	D- Lumina_HIV_AJ0	admin	05/16/2015	

## Entering Cassette Information

1. Navigate to **Order Management** > **Sample Orders**.

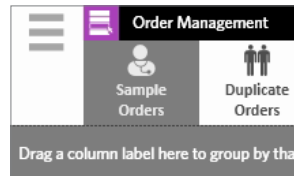

2. Scan the **Sample ID** (the system automatically finds the order) or select an order from the **Sample Orders** page.

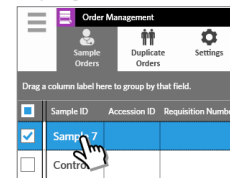


3. Scan or enter the cassette information.




OR

4. Select **Edit Order** from the Page Action bar.

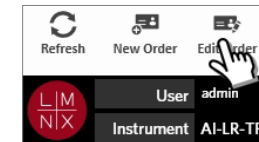

5. Scan or enter the cassette information.

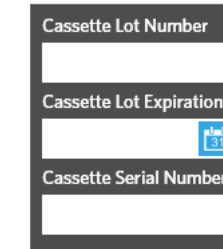


6. Select or scan **Save**.



## Starting a Run

1. Enter your **User name** and **Password**. Select **Login**.

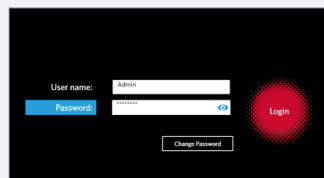


2. Load the sample into the cassette.

**NOTE:** Some assays require a pre-processing step. Refer to the package insert.




3. Load the cassette into the magazine.
4. Load the magazine into the **ARIES®** instrument.

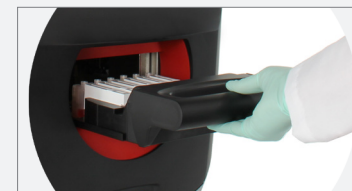

5. The **ARIES®** instrument automatically scans the cassettes. **NOTE:** If Auto Run is enabled, you do not need to continue to step 6.




6. If the scan is successful, select **Start Run** from the Page Action bar.

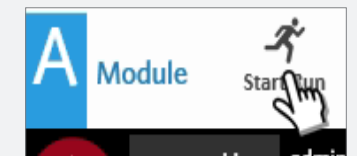

## Viewing, Printing, and Exporting Results after a Run


1. To view results, navigate to **Results** > **Results** page.

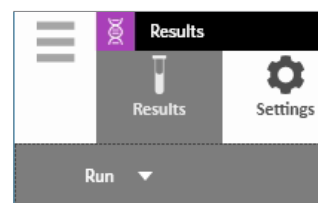


2. To print results, select only one result from the **Results** page.

Sample ID	Status	Location
AI-LR-TP1-E7064-Run#027-A (		
Sample 1		A 1
Sample 2		A 2


3. Select **Create Report** from the Page Action bar and choose a report.

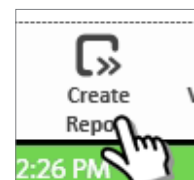

4. Once the report is generated, select **Print Report** from the Page Action bar.

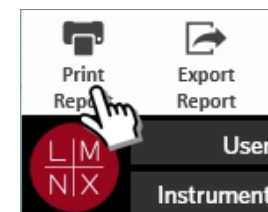

5. To export a PDF of the report, select **Export Report** from the Page Action bar.

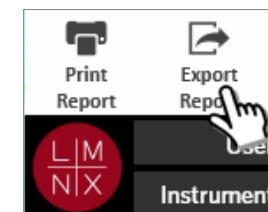

6. Choose the **Location** for the export. Select **OK**.

