

# SYPHILIS VIRCLIA® IgG+IgM MONOTEST

For *in vitro* diagnostic use

**VCM086:** Indirect chemiluminescent immunoassay (CLIA) to test IgG+IgM antibodies against *Treponema pallidum* in human serum/plasma. 24 tests.

## INTRODUCTION:

Syphilis is still a common sexually transmitted disease in many areas of the world. In 1999 the WHO estimated that the worldwide annual incidence of sexually acquired syphilis was 12 million cases. Venereal syphilis is divided into: early syphilis subdivided into primary, secondary and early latent stages; late syphilis may occur after extended periods of latent syphilis. Serological tests for syphilis include non treponemal tests that measure IgM and IgG antibodies to lipoidal material released from damaged host cells and antibodies to lipoprotein-like material and cardiolipin released from the treponemes. The most commonly used are RPR card and VDRL. The tests are used for screening and for determining the efficacy of treatment. They lack sensitivity in early primary syphilis and in late syphilis and it can appear a prozone reaction or false positive results. On the other hand, treponemal tests use *Treponema pallidum* subsp. *pallidum* or its derivatives (recombinant proteins). They are used as confirmatory tests and in establishing the diagnosis of late latent or late syphilis. The most commonly used tests are: FTA-ABS, TP-PA (*T. pallidum* particle agglutination) and TPHA (micro hemagglutination assay to *T. pallidum*). Several treponemal tests using enzyme immunoassays (EIA) have been used as confirmatory tests for syphilis and more recently as screening tests. They have sensitivities and specificities similar to those of the other treponemal tests.

Detection methods based on chemiluminescence have received much attention due to their low background, linearity and wide dynamic range. When coupled to enzyme immunoassays, the signal amplification effect provided by the enzyme enables the design of CLIA (ChemiLuminescent ImmunoAssay) tests with shorter incubation times while keeping or improving their sensitivity.

## PRINCIPLE OF THE TEST:

The CLIA method is based upon the reaction of antibodies in the sample tested with the antigen adsorbed on the polystyrene surface. Unbound immunoglobulins are washed off. An enzyme-labelled anti-human globulin binds the antigen-antibody complex in a second step. After a new washing step, bound conjugate is developed with the aid of a chemiluminescent substrate solution that will generate a glow-type luminescence that can be read with a luminometer.

## KIT FEATURES:

All reagents supplied are ready to use.  
Serum dilution solution and conjugate are coloured to help in the performance of the technique.  
Sample predilution is not necessary.  
Reagents required for the run of the test are included in the monodose presentation.

## KIT CONTENTS:

1 VIRCLIA® SYPHILIS IgG+IgM MONODOSE: 24 monodoses consisting of 3 reaction wells and 5 reagent wells with the following composition:

Wells A, B, C: reaction wells; wells coated with *T. pallidum* (Tpp15, Tpp17, Tpp47) antigen.

Well D: Conjugate: orange; containing anti-human IgG+IgM peroxidase conjugate dilution and Neolone and Bronidox as preservatives.

Well E: Serum dilution solution: blue; phosphate buffer containing protein stabilizers and Neolone and Bronidox as preservatives.

Well F: Calibrator: clear; positive serum dilution containing Neolone and Bronidox as preservative.

Well G: Substrate component B: clear; containing peroxide.

Well H: Substrate component A: clear; containing luminol.

**Store at 2-8°C and check expiration date.**

## Materials required but not supplied:

-VIRCLIA® AUXILIARY REAGENTS (REF:VCMAR)

-Precision micropipettes 5 and 100 µl.

-Eight channel micropipette 100 µl.

-Adapted microplate washer.

-Thermostated incubator/water bath.

-Microplate luminometer.

-Alternatively, a CLIA automated processor.

## STORAGE REQUIREMENTS:

Store at 2-8°C. Do not use the kit reagents beyond the expiration date. This will be valid only if reagents are stored closed and at 2-8°C.

## STORAGE OF REAGENTS ONCE OPENED:

Reagent	Stability
VIRCLIA® MONODOSE	Once opened, use it in the same day

## STABILITY AND HANDLING OF REAGENTS:

Handle reagents in aseptic conditions to avoid microbial contaminations.

Do not let the plate dry between washing and reagent addition.

Substrate component A is light sensitive. Avoid light exposure. Substrate solution should not get in contact with acid, combustible materials and strong oxidizing or reducing agents. Make sure that no metal components come in contact with the substrate without having previously tested their compatibility.

VIRCELL, S.L does not accept responsibility for the mishandling of the reagents included in the kit.

## RECOMMENDATIONS AND PRECAUTIONS:

- For *in vitro* diagnosis use only. For professional use only.
- Use kit components only. Do not mix components from different kits or manufacturers. Only components of the AUXILIARY REAGENTS kit are compatible with all VIRCLIA® references and lots.
- Clean pipette tips must be used for every assay step. Use only clean, preferably disposable material.
- Wear protective disposable gloves, laboratory coats and eye protection when handling specimens. Wash hands thoroughly



after manipulating samples. Besides, follow all safety protocols in use in your laboratory.

5. Do not use in the event of damage to the package.
6. Never pipette by mouth.
7. Serum dilution solution, reaction wells, conjugates and calibrator in this kit include substances of animal origin. Calibrator includes as well substances of human origin. Although the human serum controls of this kit have been tested and found negative for Hepatitis B Surface Antigen (HBsAg), Hepatitis C antibodies and Human Immunodeficiency Virus antibodies, control sera and patient specimens should be handled as potentially infectious. Reaction wells are coated with inactivated antigen. Nevertheless, they should be considered potentially infectious and handled with care. No present method can offer complete assurance that infectious agents are absent. All material should be handled and disposed as potentially infectious. Observe the local regulations for clinical waste disposal.
8. Substrate solution may be irritant to eyes, respiratory system and skin. In case of contact with this solution, rinse thoroughly with water and seek medical attention. For further information a Material Safety Data Sheet is available.
9. Do not use this product in automated processors unless they have been previously validated for that purpose.

#### SPECIMEN COLLECTION AND HANDLING:

Blood should be collected aseptically using venipuncture techniques by qualified personnel. Use of sterile or aseptic techniques will preserve the integrity of the specimen. Serum samples are to be refrigerated (2-8°C) upon collection or frozen (-20°C) if the test cannot be performed within 7 days. Samples should not be repeatedly frozen and thawed. Do not use hyperlipemic, hemolysed or contaminated sera. Samples containing particles should be clarified by centrifugation. The kit is suitable for use with serum or plasma.

#### PRELIMINARY PREPARATION OF THE REAGENTS:

All reagents supplied are ready to use.

Only the VIRCLIA® WASHING SOLUTION included in the auxiliary component kit VIRCLIA® AUXILIARY REAGENTS must be prepared in advance. Fill 50 ml of VIRCLIA® WASHING SOLUTION (20x) up to 1 litre with distilled water. Should salt crystals form in the washing concentrate during storage, warm the solution to 37°C before diluting. Once diluted, store at 2-8°C.

#### ASSAY PROCEDURE:

##### • AUTOMATED

1. Bring VIRCLIA® WASHING SOLUTION (diluted according to the instructions) to room temperature before use (approximately 1 hour).

2. Follow the Operator's Manual of the Automated Processor.

##### • MANUAL

1. Set incubator/water bath to 37±1°C.

2. Bring VIRCLIA® WASHING SOLUTION (diluted according to the instructions) to room temperature before use (approximately 1 hour).

3. Remove the monodoses from the package and let them reach room temperature before use (approximately 20 minutes). Determine the numbers of monodoses to be employed (one for every sample to be tested).

4. Place the strips into a frame. With the aid of a clean pipette tip, puncture the foil of the well in position E (see the left side of the frame for reference), draw 100 µl of reagent (sample

diluent) and dispense it into the white well in position B. Add 5 µl of sample into the white well in position B. Mix homogeneously with the aid of a pipette. With the aid of a clean pipette tip, puncture the foil of the well in position F, draw 100 µl of reagent (calibrator) and dispense it into white well in position A. Draw 80 µl of reagent from well E with a new clean tip and add it into the white well in position C, then draw 20 µl of reagent from well F and add it into the white well in position C. Mix homogeneously with the aid of a pipette.

5. Incubate at 37±1°C for 20 minutes.

6. Aspirate liquid from wells A, B, C and wash five times with 0.3 ml of VIRCLIA® WASHING SOLUTION (diluted according to the instructions) per well. Drain off any remaining liquid.

7. Immediately add 50 µl of VIRCLIA® WASHING SOLUTION (diluted according to the instructions) into each one of wells A, B, C.

8. With the aid of a clean pipette tip, puncture the foil of the well in position D, draw 50 µl of reagent (conjugate) and dispense it into the white well in position A. Repeat this step for wells B and C.

9. Incubate at 37±1°C for 20 minutes.

10. Aspirate liquid from wells A, B, C and wash five times with 0.3 ml of VIRCLIA® WASHING SOLUTION (diluted according to the instructions) per well. Drain off any remaining liquid.

11. With the aid of a clean pipette tip, puncture the foil of the well in position H, draw 50 µl of reagent (substrate component A) and dispense it into the white well in position A. Repeat this step for wells B and C.

12. With the aid of a clean pipette tip, puncture the foil of the well in position G, draw 50 µl of reagent (substrate component B) and dispense it into the white well in position A. Repeat this step for wells B and C. Mix gently for 10 seconds with the aid of a microtiter plate shaker or by gently tapping on the sides of the plate.

13. Incubate at 37°C for 5 minutes protected from light.

14. Measure relative luminescence units (RLU) in wells A, B, C with the help of a luminometer.

#### INTERNAL QUALITY CONTROL:

Each batch is subjected to internal quality control (Q.C.) testing before batch release complying with specifications stricter than validation protocol for users. Final Q.C. results for each particular lot are available.

The control material is traceable to reference sera panels internally validated.

#### VALIDATION PROTOCOL FOR USERS:

Each monodose includes one calibrator (well A) and one dilution of the calibrator used as negative control (well C). It allows the validation of the assay and kit.

RLU of the calibrator and the negative control must fall in the following ranges. Otherwise, the test is invalid and must be repeated.

Control	RLU
CALIBRATOR	2-7
NEGATIVE CONTROL	<2



**INTERPRETATION OF RESULTS:**

Antibody index= (sample RLU/calibrator RLU)

Index	Interpretation
<0.9	Negative
0.9-1.1	Equivocal
>1.1	Positive

Samples with equivocal results must be retested and/or a new sample obtained for confirmation.

Samples with indexes below 0.9 are considered as not having antibodies of the specificity and class measured by this kit.

Samples with indexes above 1.1 are considered as having antibodies of the specificity and class measured by this kit.

**LIMITATIONS:**

1. This kit is intended to be used with human serum/plasma.
2. The user of this kit is advised to carefully read and understand the package insert. Strict adherence to the protocol is necessary to obtain reliable test results. In particular, correct sample and reagent pipetting, along with careful washing and timing of the incubation steps are essential for accurate results.
3. The results of samples should be used in conjunction with clinical evaluation and other diagnostic procedures. A definitive diagnosis should be made by isolation techniques.
4. This test will not indicate the site of infection. It is not intended to replace isolation.
5. Lack of significant rise in antibody level does not exclude the possibility of infection.
6. Samples collected very early in the course of an infection may not have detectable levels of IgG. In such cases, it is recommended an IgM assay be performed or a second serum sample be obtained 14 to 21 days later to be tested in parallel with the original sample to determine seroconversion.
7. Results in IgG detection in neonates must be interpreted with caution, since maternal IgG is transferred passively from the mother to the foetus before birth. IgM assays are generally more useful indicators of infection in children below 6 months of age.
8. The results of a single-specimen antibody determination should not be used to aid in the diagnosis of recent infection. Paired samples (acute and convalescent) should be collected and tested concurrently to look for seroconversion or a significant rise in antibody level.
9. The performance results showed correspond to comparative studies with commercial predicative devices in a defined population sample. Small differences can be found with different populations or different predicative devices.

**PERFORMANCES:****• SENSITIVITY AND SPECIFICITY:**

138 serum/plasma samples were assayed against a commercial ELISA kit. The results were as follows:

Samples No.	Sensitivity	Specificity
138	98%	100%

Indeterminate values were omitted from the final calculations.

**• INTRA-ASSAY PRECISION:**

3 sera were individually run 10 times each serum in a single automated assay in essentially unchanged conditions.

The results were as follows:

Serum	N	% C.V.
Sample +	10	6
CAL	10	9
CN	10	9

C.V. Coefficient of variation

**• INTER-ASSAY PRECISION:**

3 sera were individually run on 5 consecutive days in 2 different automatic processors.

The results were as follows:

Serum	N	% C.V.
Sample +	10	13
CAL	10	10
CN	10	19









C.V. Coefficient of variation

**• CROSS REACTIVITY AND INTERFERENCES:**

8 samples known to be positive for other spirochetes (*Borrelia burgdorferi*) and members of the syndromic group (Herpes simplex 2, *Chlamydia trachomatis*) were assayed. 2 samples known to be positive for antinuclear antibodies were assayed. 2 samples known to be positive for rheumatoid factor were assayed.

The negative results of the test demonstrated the specific reaction of the kit with no cross reaction or interferences with the referred specimens.

**SYMBOLS USED IN LABELS:**

	In vitro diagnostic medical device
	Use by (expiration date)
	Store at x-y°C
	Contains sufficient for <n> test
	Batch code
	Catalogue number
	Consult instructions for use
	<X> wells

**BIBLIOGRAPHY:**

1. Coles, F. B., S. S. Hipp, G. S. Silberstein, and J. H. Chen. 1995. Congenital syphilis surveillance in upstate New York, 1989-1992: implications for prevention and clinical management. *J Infect Dis* 171:732-5.
2. Fujimura, K., N. Ise, E. Ueno, T. Hori, N. Fujii, and M. Okada. 1997. Reactivity of recombinant *Treponema pallidum* (r-Tp) antigens with anti-Tp antibodies in human syphilitic sera evaluated by ELISA. *J Clin Lab Anal* 11:315-22.
3. Gerber, A., S. Krell, and J. Morenz. 1996-1997. Recombinant *Treponema pallidum* antigens in syphilis serology. *Immunobiology* 196:535-49.



4. Halling, V. W., M. F. Jones, J. E. Bestrom, A. D. Wold, J. E. Rosenblatt, T. F. Smith, and F. R. Cockerill 3rd. 1999. Clinical comparison of the *Treponema pallidum* CAPTIA syphilis-G enzyme immunoassay with the fluorescent treponemal antibody absorption immunoglobulin G assay for syphilis testing. *J Clin Microbiol* 37:3233-4.
5. Hook, E. W. 3rd. 1994. Diagnosing neurosyphilis. *Clin Infect Dis* 18:295-7.
6. Larsen, S. A., B. M. Steiner, and A. H. Rudolph. 1995. Laboratory diagnosis and interpretation of tests for syphilis. *Clin Microbiol Rev* 8:1-21.
7. Marangoni, A., V. Sambri, E. Storni, A. D'Antuono, M. Negosanti, and R. Cevenini. 2000. *Treponema pallidum* surface immunofluorescence assay for serologic diagnosis of syphilis. *Clin Diagn Lab Immunol* 7:417-21.
8. Merlin, S., J. Andre, B. Alacoque, and A. Paris-Hamelin. 1985. Importance of specific IgM antibodies in 116 patients with various stages of syphilis. *Genitourin Med* 61:82-7.
9. Moskophidis, M. and F. Muller. 1984. Molecular analysis of immunoglobulins M and G immune response to protein antigens of *Treponema pallidum* in human syphilis. *Infect Immun* 43:127-32.
10. Pedersen, N. S., J. P. Sheller, A. V. Ratnam, and S. K. Hira. 1989. Enzyme-linked immunosorbent assays for detection of immunoglobulin M to nontreponemal and treponemal antigens for the diagnosis of congenital syphilis. *J Clin Microbiol* 27:1835-40.
11. Radolf, J. D., E. B. Lernhardt, T. E. Fehniger, and M. A. Lovett. 1986. Serodiagnosis of syphilis by enzyme-linked immunosorbent assay with purified recombinant *Treponema pallidum* antigen 4D. *J Infect Dis* 153:1023-7.
12. Sambri, V., A. Marangoni, M. A. Simone, A. D'Antuono, M. Negosanti, and R. Cevenini. 2001. Evaluation of recomWell *Treponema*, a novel recombinant antigen-based enzyme-linked immunosorbent assay for the diagnosis of syphilis. *Clin Microbiol Infect* 7:200-5.
13. Stoll, B. J. 1994. Congenital syphilis: evaluation and management of neonates born to mothers with reactive serologic tests for syphilis. *Pediatr Infect Dis J* 13:845-52; quiz 853.
14. Tomberlin, M. G., P. D. Holtom, J. L. Owens, and R. A. Larsen. 1994. Evaluation of neurosyphilis in human immunodeficiency virus-infected individuals. *Clin Infect Dis* 18:288-94.
15. Van Voorhis, W. C., L. K. Barrett, S. A. Lukehart, B. Schmidt, M. Schriefer, and C. E. Cameron. 2003. Serodiagnosis of syphilis: antibodies to recombinant Tp0453, Tp92, and Gpd proteins are sensitive and specific indicators of infection by *Treponema pallidum*. *J Clin Microbiol* 41:3668-74.
16. Young, H., G. Aktas, and A. Moyes. 2000. Enzywell recombinant enzyme immunoassay for the serological diagnosis of syphilis. *Int J STD AIDS* 11:288-91.
17. Young, H., A. Moyes, A. McMillan, and J. Patterson. 1992. Enzyme immunoassay for anti-treponemal IgG: screening or confirmatory test? *J Clin Pathol* 45:37-41.
18. Velan, B., M. Halmann. 1978. Chemiluminescence immunoassay. A new sensitive method for determination of antigens. *Immunochemistry*. 15: 331-333.
19. Whitehead, T.P., L.J. Kricka, T.J. Carter, G.H. Thorpe. 1979. Analytical luminescence: its potential in the clinical laboratory. *Clin Chem*. 25:1531-46.
20. Zhao, L., L. Sun, X. Chu. 2009. Chemiluminescence immunoassay. *TrAC-Trend Anal Chem*. 28: 404-415.

For any questions please contact:  
[customerservice@vircell.com](mailto:customerservice@vircell.com)

REVISED: 2016/03

